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Please read the instructions completely before you start taking the samples! If you have any questions: please contact your practitioner first!

The test kit contains:

- Instruction manual for fecal sample collection
- 2 x Stool collection tube (brown cap)
- 1 x TFT kit (3 x stool collection tube large)
DAY 1 and DAY 3 contain (SAF) liquid,
for fixation of parasites

Please note: The fixation liquid (SAF) has a low toxicity. Skin contact is not harmful when rinsed off immediately with water. The warning is mainly intended to prevent children from drinking the liquid. Keep out of reach of children. **Do not pour the fixation liquid out of the collection tube.**

- 2 x TRP/HIS tube (yellow/blue cap with stabilizer)

Please note: The stabilizer is a corrosive liquid, do not bring into contact with skin. Keep out of reach of children. **Do not pour the stabilizer out of the collection tube.**

- 2 x Label for TRP/HIS tube
- 3 x Feces catcher
- 2 x RigidSafetyBag®
- Safetybag

You also received a return envelope and application form from RPSH Analytic. Is the test kit incomplete? Please contact us by phone at +31(0)113-370170 (Mon–Fri 8:30–12:30 and 13:30–17:00) or by e-mail: info@rps.nl.

Preparations

1. Fill in your name and date of birth on the label of the stool collection tubes.
2. Fill in your name and date of birth on the labels of the TRP/HIS tubes and stick it on the TRP/HIS tubes.
3. Check if your personal details have been filled in correctly on the application form.

4. Check if there is liquid in the TRP/HIS tubes and stool collection tubes 1 and 3 of the TFT-kit. If not, please contact us first on the aforementioned details.
5. Fill in the date of the sample collection on the application form on each collection day, and agree to the terms and conditions.
6. Watch the following instructional videos by scanning the QR-code:

Or watch via the website:
www.rpsh.de/rpsh-analytic/testkits

E

Important!

- The TFT-kit must be collected on **three consecutive days**. Intestinal parasites are excreted irregularly, so it is important to take stool samples on three different days.
- If you have multiple bowel movements per day, you should still only fill a collection tube once per day. If you do not have daily bowel movements, wait until you have filled all three collection tubes. The extra days between sample collections will not affect the test.
- The stool collection and histamine tubes should be collected together with the last tube of the TFT-kit on DAY 3.
- The fixation liquid in the TFT-kit (collection tubes 1 and 3) is needed to detect the so-called 'vegetative stages' of intestinal parasites. After the bowel movements on DAY 1 and DAY 3, the stool should be placed directly in the stool collection tubes, because some stages of the intestinal parasites are no longer detectable outside the body after 30 minutes.
- With flush toilets, the stool falls directly into the water. This makes collection difficult and can reduce the reliability of the test. In addition to the feces catcher, it is recommended to place a few longer pieces of toilet paper folded in half on the surface of the water. The stool can then be collected from the feces catcher or the toilet paper.


Additional guidelines for women

Please do not collect stool during menstruation.

Sample collection can be started from Saturday. Send from Monday to Thursday at the latest (NL). Please note: send the test kit immediately on the last day of collection.


Sample collection DAY 1 TFT-kit

1. Attach the feces catcher to the toilet seat and collect your stool from it.
2. Take TFT collection tube 1 with liquid from the blister and fill the collection tube with stool.
Please note: The stool and liquid together may not exceed the red line. Make sure that the stool is completely covered by the liquid.
3. After filling it, screw the cap straight and firmly onto the collection tube and shake the tube gently (approx. 20 seconds).
4. Write the collection date on the collection tube and the application form.
5. Store the collection tube in the blister in the refrigerator.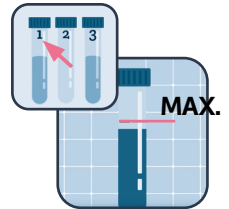



Sample collection DAY 2 TFT-kit

1. Attach the feces catcher to the toilet seat and collect your stool from it.
2. Take TFT collection tube 2 without liquid from the blister and fill the collection tube halfway with stool.
Please note: Take a scoop of stool from at least 3 different places!
3. After filling it, screw the cap straight and firmly onto the collection tube.
4. Write the collection date on the collection tube and the application form.
5. Store the collection tube in the blister in the refrigerator.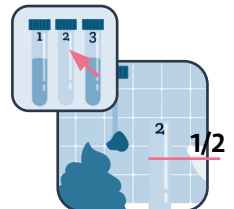

Sample collection DAY 3 TFT-kit


1. Attach the feces catcher to the toilet seat and collect your stool from it.
2. Take TFT collection tube 3 with liquid from the case and fill the collection tube with stool.
Please note: The stool and liquid together may not exceed the red line. Make sure that the stool is completely covered by the liquid.
3. After filling it, screw the cap straight and firmly onto the collection tube and shake the tube gently (approx. 20 seconds).
4. Write the collection date on the collection tube and the application form.
5. Place the third TFT collection tube in the case and put the case with the 3 collection tubes in the Safetybag.
6. Remove the white foil from the Safetybag. Press the exposed adhesive strip so that the bag is properly sealed.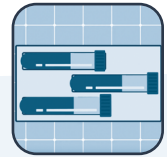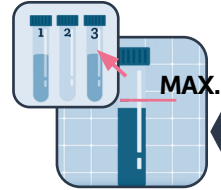

Stool collection tubes

7. Fill the two stool collection tubes at least halfway with stool.
Please note: Take a scoop of stool from at least 6 different places.
8. Close the stool collection tubes and place them in the RigidSafetyBag® in the hard plastic cover. Press the exposed adhesive strip so that the bag is properly sealed.

– *The RigidSafetyBag® contains absorbent material that protects the collection tubes during transport.*

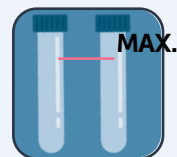Shipping

NL: RPSH Analytic, antwoordnummer 1461, 4460 VS Kamperland

Belgium: RPSH Analytic, drop off at an UPS Access Point

Germany: RPSH Analytic, Veerweg 16, 4493 AS Kamperland – Niederlande (Per Express)

Other countries: please contact us at info@rps.nl

Please note: Do not send during the weekend or the day before a national holiday!

NL: Within the Netherlands, send from **Monday to Thursday** (from an orange PostNL postbox or PostNL-location).

1. Check whether the postbox has already been emptied.
2. Use a medical postbox if the regular postbox is already empty. These postboxes can be recognised by the logo 'medische post'.
3. Find a medical postbox via the PostNL Location Finder, filter on 'post a letter' and select 'medical' at the bottom.

Scan the QR code for the locations of the medical postboxes:

<https://www.postnl.nl/en/locationfinder/>   **MEDISCHE POST**

Please note: If the test kit is not sent via a medical postbox, but via a PostNL point, it will be treated as a parcel. This may delay delivery and may prevent the test kit from being processed due to the quality of the sample.

Belgium: From Belgium send from **Monday to Wednesday** via UPS.

- After taking the test kit, the patient can drop the sealed return envelope off at one of the UPS Access Points (do NOT deposit it in a Bpost mailbox!)
- Send from Monday to Wednesday (this guarantees that the test kit can be processed before the weekend)
- The cost for the return shipment is paid by RPSH.

The UPS Access Points can be found on the UPS website under 'Locations' at the top right [ups.com/dropoff/?loc=nL_BE]

Please note: The UPS Access Point you select will indicate the latest collection time for 'Standard' service. The test kit must be dropped off before this time to be delivered to RPSH Analytic the next day!