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
Please read the instructions completely before you start taking the samples! If you have any questions: please contact your practitioner first!

If the Saliva 7-test kit is also required, the Saliva 3-test kit is not necessary.

The test kit contains:

- Instruction manual for saliva sample collection
- 3 x Saliva straws
- 3 x Collection tubes
- 3 x Collection tube labels
- Shipping case
- Safetybag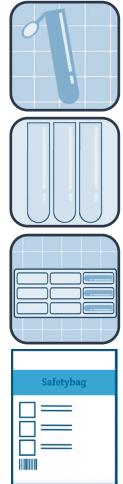

You also received a return envelope and application form from RPSH Analytic. Is the test kit incomplete? Please contact us by phone at +31(0)113-370170 (Mon-Fri 8:30-12:30 and 13:30-17:00) or by e-mail: info@rps.nl.

Preparations

1. Fill in your name and date of birth on the labels of the 3 collection tubes.
2. Check if your personal details have been filled in correctly on the application form.
3. Fill in the date of the sample collection on the application form and agree to the terms and conditions.
5. Watch the following instructional video by scanning the QR-code:

Or watch via the website:
<https://www.rps.nl/rps-analytic/testkits/speeksel/>

Important!

When scheduling your test (see later on), please consider the following:

- Brush your teeth only after filling the third collection tube. Wait 30 minutes after brushing before collecting a saliva sample.
- The test must be sent the same day. Refrigerate the collection tube immediately after collection.
- Do not use mouthwash on the day of collection.
- Do not eat breakfast until after completing the test.
- Avoid blood in the saliva sample. To prevent bleeding gums, we recommend not using toothpicks or dental floss on the day of collection.
- **Please note:** Do not take or send this test in weather conditions above 25 °C.

Are you taking any medications or supplements?

- Stop using hormone preparations (tablets, capsules, creams, or gels; hydrocortisone or progesterone) 48 hours before the saliva collection.
- Stop taking nutritional supplements that affect hormonal balance (e.g. Maca) 5 days before the saliva collection. If in doubt, please consult your practitioner.

Additional guidelines for women

- If you are taking the contraceptive pill, you can take the test at the beginning of your stop week. If you skip the stop week, you can postpone your pill intake (in the week before the test) to the afternoon. This will allow more than 12 hours between taking the pill and the saliva sample.
- If you are not taking the contraceptive pill, it is best to take the test between the 19th and 21st day of your cycle, if your total cycle length is 26 to 29 days (the first day of your cycle is the first day of menstruation). If your cycle is shorter than 26 days or longer than 29 days, or if you have irregular periods, contact your practitioner.

**Collect and send from Monday through Thursday.
Preferably, send the test kit on the day of collection.**


Sample collection

Use your daily routine as a guide for the saliva collection.


Set a timer to ensure you fill the next collection tube on time.

You may deviate from the schedule by a maximum of 10 minutes (see next page).

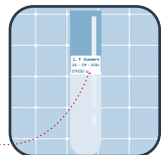1. Rinse your mouth with water immediately before collecting the saliva sample. Spit out the water or swallow it.
2. Take a saliva straw and the collection tube.
Tip: Fill in the time schedule on the next page with the times from waking up*.
3. Allow the amount of saliva to increase in your mouth. Place the saliva straw between your lips and spit your saliva through the straw into the collection tube. Fill the tube halfway with saliva. *Foam is fine, but note that this does not count as tube filling!*
4. Repeat step 3 until the collection tube is halfway full.
5. Close the lid of the collection tube. Store the straw in a plastic bag until the entire test has been successfully completed.
6. Fill in the collection time on the label and attach it to the tube.
7. Place the collection tube in the shipping case and store it in the refrigerator.
8. Set an alarm on your phone for the next collection time.
9. Repeat steps 1 through 7 for the next collection. Check the schedule for the correct times.

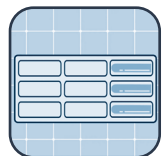L. T. Somers
22 - 09 - 2021
07:00 u

Sample collection schedule

Sample number	color tube	Collection moment	Fill in collection time	Note
1	 Transparent	Directly after waking : o'clock	Do not: eat, drink, smoke, brush teeth
2	 Transparent	30 min. after waking : o'clock	Do not: eat, drink, smoke, brush teeth
3	 Transparent	1 uur after waking : o'clock	Do not: eat, drink, smoke, brush teeth

* Times may vary by a maximum of 10 minutes.

Tip: Use the alarm on your mobile phone as a reminder for the next saliva sample.

Other actions

- Place the shipping case containing the three collection tubes in the Safetybag.
Press the exposed adhesive strip to ensure the bag is properly sealed.
- Check the collection date on the application form and whether you have agreed to the terms and conditions.
- Place the Safetybag and the application form in the return envelope and seal it. The return envelope is now ready to be send to our laboratory. (Keep the return envelope in the refrigerator until posting).

Retaking the test

If something went wrong during the test, you can retake the test on another day (for example: blood in saliva, missed test tubes, 10 minutes late according to schedule).

1. On the day of the failed test: carefully rinse the used collection tubes and saliva straws under the tap with lukewarm water. If there is no longer saliva residue visible, shake off as much water as possible and let the materials dry.
2. The evening before you attempt the next try: check that the materials are thoroughly dry. If any water droplets are still visible, remove them by wiping them with a clean, dry cotton swab.
3. On the day of the next try: change the collection date (on application form and labels). If you woke up at a different time, change the times as well.

Shipping

NL: RPSH Analytic, antwoordnummer 1461, 4460 VS Kamperland

Belgium: RPSH Analytic, drop off at an UPS Access Point

Germany: RPSH Analytic, Veerweg 16, 4493 AS Kamperland – Niederlande (Per Express)

Other countries: please contact us at info@rpsn.nl

Please note: Do not send during the weekend or the day before a national holiday!

NL: Within the Netherlands, send from **Monday to Thursday** (from an orange PostNL postbox).

1. Check whether the postbox has already been emptied.
2. Use a medical postbox if the regular postbox is already empty. These postboxes can be recognised by the logo 'medische post'.
3. Find a medical postbox via the PostNL Location Finder, filter on 'post a letter' and select 'medical' at the bottom.

Scan the QR code for the locations of the medical postboxes:

<https://www.postnl.nl/en/locationfinder/>   **MEDISCHE POST**

Please note: If the test kit is not sent via a medical postbox, but via a PostNL point, it will be treated as a parcel. This may delay delivery and may prevent the test kit from being processed due to the quality of the sample.

Belgium: From Belgium send from **Monday to Wednesday** via UPS.

- After taking the test kit, the patient can drop the sealed return envelope off at one of the UPS Access Points (do NOT deposit it in a Bpost mailbox!)
- Send from Monday to Wednesday (this guarantees that the test kit can be processed before the weekend)
- The cost for the return shipment is paid by RPSH.

The UPS Access Points can be found on the UPS website under 'Locations' at the top right [ups.com/dropoff/?loc=nL_BE]

Please note: The UPS Access Point you select will indicate the latest collection time for 'Standard' service. The test kit must be dropped off before this time to be delivered to RPSH Analytic the next day!

RPSH
Sanitas Humanus