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Please read the instructions completely before you start taking the samples! If you have any questions: please contact your practitioner first!

The test kit contains:

- Instruction manual for fecal sample collection
- TRP/HIS tube (yellow/blue cap with stabilizer)
Please note: The stabilizer is a corrosive liquid, do not bring into contact with skin. Keep out of reach of children. **Do not pour the stabilizer out of the collection tube.**
- Label for TRP/HIS tube
- Feces catcher
- RigidSafetyBag®

E

You also received a return envelope and application form from RPSH Analytic. Is the test kit incomplete? Please contact us by phone at +31(0)113-370170 (Mon–Fri 8:30–12:30 and 13:30–17:00) or by e-mail: info@rps.nl.

Preparations

1. Fill in your name and date of birth on the label of the TRP/HIS tube and stick it on the TRP/HIS tube.
2. Check if your personal details have been filled in correctly on the application form.
3. Check if there is liquid in the TRP/HIS tube. If not, please contact us first on the aforementioned details.
4. Fill in the date of the sample collection on the application form on each collection day, and agree to the terms and conditions.
5. Watch the following instructional video by scanning the QR-code:


Histamine

Or watch via the website:
www.rps.nl/rps-analytic/testkits/histamine/

5. Close the collection tube tightly.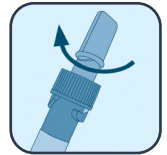

6. Place the TRP/HIS tube in the hard plastic case.

7. Remove the white foil from the RigidSafetyBag®. Press the exposed adhesive strip so that the bag is properly sealed.
– The RigidSafetyBag® contains absorbent material that protects the tube during transport.

8. Please check the sample collection date on the application form and whether you have agreed to the terms and conditions.

9. Place the Safetybag together with the application form in the return envelope and seal it. The return envelope is now ready to be send to our laboratory.

Please note: If both Tryptophan and Histamine are requested, two separate collection tubes must be submitted. Both tests (biomarkers) cannot be conducted from a single tube.

Shipping

NL: RPSH Analytic, antwoordnummer 1461, 4460 VS Kamperland

Belgium: RPSH Analytic, drop off at an UPS Access Point

Germany: RPSH Analytic, Veerweg 16, 4493 AS Kamperland – Niederlande (Per Express)

Other countries: please contact us at info@rpsn.nl

Please note: Do not send during the weekend or the day before a national holiday!

NL: Within the Netherlands, send from **Monday to Thursday** (from an orange PostNL postbox).

1. Check whether the postbox has already been emptied.
2. Use a medical postbox if the regular postbox is already empty. These postboxes can be recognised by the logo 'medische post'.
3. Find a medical postbox via the PostNL Location Finder, filter on 'post a letter' and select 'medical' at the bottom.

Scan the QR code for the locations of the medical postboxes:

<https://www.postnl.nl/en/locationfinder/>

Please note: If the test kit is not sent via a medical postbox, but via a PostNL point, it will be treated as a parcel. This may delay delivery and may prevent the test kit from being processed due to the quality of the sample.

Belgium: From Belgium send from **Monday to Wednesday** via UPS.

- After taking the test kit, the patient can drop the sealed return envelope off at one of the UPS Access Points (do NOT deposit it in a Bpost mailbox!)
- Send from Monday to Wednesday (this guarantees that the test kit can be processed before the weekend)
- The cost for the return shipment is paid by RPSH.

The UPS Access Points can be found on the UPS website under 'Locations' at the top right [[ups.com/dropoff/?loc=nL_BE](https://www.ups.com/dropoff/?loc=nL_BE)]

Please note: The UPS Access Point you select will indicate the latest collection time for 'Standard' service. The test kit must be dropped off before this time to be delivered to RPSH Analytic the next day!

RPSH
Sanitas Humanus