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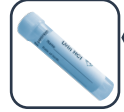
Please read the instructions completely before you start taking the samples! If you have any questions: please contact your practitioner first!

### The test kit contains:

- Instruction manual for urine sample collection
- Urine tube with stabilizer (hydrochloric acid) + label  
**Caution:** The stabilizer is a corrosive liquid; do not allow it to come into contact with skin. Keep out of reach of children. Do not pour the stabilizer out of the tube.
- Urine cup
- pH test strip
- Shipping tube
- Safetybag

E

You also received a return envelope and application form from RPSH Analytic. Is the test kit incomplete? Please contact us by phone at +31(0)113-370170 (Mon–Fri 8:30–12:30 and 13:30–17:00) or by e-mail: [info@rps.nl](mailto:info@rps.nl).

### Preparations

1. Fill in your name and date of birth on the labels of the urine tube.
2. Check if your personal details have been filled in correctly on the application form.
3. Fill in the date of the sample collection on the application form and agree to the terms and conditions.

### Important!

- Use the pH-test strip only after you have poured the urine into the tube with stabilizer.

## Important:

If the pH value is **NOT between 0 and 3.0**:

- Do **NOT send in** the sample
- Consult your healthcare practitioner
- Request a new test kit

9. Close the urine tube and check the information on the label.
10. Put the urine tube in the shipping tube and place it in the Safetybag.
11. Remove the white foil from the Safetybag. Press the exposed adhesive strip so that the bag is properly sealed.
12. Check the collection date on the request form and whether you have agreed to the terms and conditions.
13. Place the Safetybag together with the application form in the return envelope and seal it. The return envelope is now ready to be sent to our laboratory.

## Shipping

**NL: RPSH Analytic**, antwoordnummer 1461, 4460 VS Kamperland

**Belgium: RPSH Analytic**, drop off at an UPS Access Point

**Germany: RPSH Analytic**, Veerweg 16, 4493 AS Kamperland – Niederlande (Per Express)

**Other countries:** please contact us at [info@rpsn.nl](mailto:info@rpsn.nl)

**Please note: Do not send during the weekend or the day before a national holiday!**

**NL:** Within the Netherlands, send from **Monday to Thursday** (from an orange PostNL postbox).

1. Check whether the postbox has already been emptied.
2. Use a medical postbox if the regular postbox is already empty. These postboxes can be recognised by the logo 'medische post'.
3. Find a medical postbox via the PostNL Location Finder, filter on 'post a letter' and select 'medical' at the bottom.

Scan the QR code for the locations of the medical postboxes:

<https://www.postnl.nl/en/locationfinder/>

**Please note:** If the test kit is not sent via a medical postbox, but via a PostNL point, it will be treated as a parcel. This may delay delivery and may prevent the test kit from being processed due to the quality of the sample.

**Belgium:** From Belgium send from **Monday to Wednesday** via UPS.

- After taking the test kit, the patient can drop the sealed return envelope off at one of the UPS Access Points (do NOT deposit it in a Bpost mailbox!)
- Send from Monday to Wednesday (this guarantees that the test kit can be processed before the weekend)
- The cost for the return shipment is paid by RPSH.

The UPS Access Points can be found on the UPS website under 'Locations' at the top right [[ups.com/dropoff/?loc=nL\\_BE](https://ups.com/dropoff/?loc=nL_BE)]

**Please note:** The UPS Access Point you select will indicate the latest collection time for 'Standard' service. The test kit must be dropped off before this time to be delivered to RPSH Analytic the next day!

**RPSH**  
Sanitas Humanus