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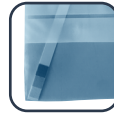
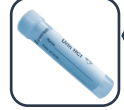
Please read the instructions completely before you start taking the samples! If you have any questions: please contact your practitioner first!

The test kit contains:

- Instruction manual for urine sample collection
- Urine tube with stabilizer (hydrochloric acid) + label
Caution: The stabilizer is a corrosive liquid; do not allow it to come into contact with skin. Keep out of reach of children. Do not pour the stabilizer out of the tube.
- Urine cup
- pH test strip
- Shipping tube
- Safetybag

E

You also received a return envelope and application form from RPSH Analytic. Is the test kit incomplete? Please contact us by phone at +31(0)113-370170 (Mon–Fri 8:30–12:30 and 13:30–17:00) or by e-mail: info@rpsn.nl.

Preparations

1. Fill in your name and date of birth on the labels of the urine tube.
2. Check if your personal details have been filled in correctly on the application form.
3. Fill in the date of the sample collection on the application form and agree to the terms and conditions.

Important!

- Use the pH-test strip only after you have poured the urine into the tube with stabilizer.

Test preparation:

1. Limit fluid intake

On the evening before and during the night prior to sampling:

- Warm weather: max. 500 ml
- Cold weather: max. 300 ml

2. Base supplementation with alkaline agents such as Basica, etc., must be stopped 24 hours before sample collection.

3. Refrain from the intake of medication, the administration of renal-permeable X-ray contrast agents, and physical activities (sports) at least 8 hours before sample collection.

4. When testing the serotonin parameter, please note the following:

4.1 To avoid falsifying the lab results, no serotonin-containing foods may be eaten 24 hours before and during the sample collection – such as bananas, walnuts, tomatoes, pineapple, gooseberries and currants, plums, mirabelles, melons, avocados, eggplants, kiwis, and cocoa.

4.2 The same applies to certain medications. If you are using any of the substances listed below, consult your therapist before stopping.

Medicines: Acetylsalicylic acid (e.g. ASS®, Aspirin®), Coumarins (e.g. Marcumar®, Venalot®), Guaifenesin (e.g. Wick Formel 44 cough suppressant, Fagusan®, Longtussin duplex®), Imipramine (e.g. Pryleugan®, Tofranil®), Isoniazid (e.g. Isozid®), Levodopa (e.g. Dopaflex®), Mephenesin (e.g. DoloVisano®), Methamphetamine (e.g. Pervitin®), Methocarbamol (e.g. Ortoton®), Paracetamol (e.g. ben-u-ron®, Captin®, Enelfa®, Grippostad®, Sinpro®, Togal®, Wick MediNait®), Phenothiazines (e.g. Atosil®, Dapotum®, Melleril®, Promethazin®), Reserpine (e.g. Briserin®, Triniton®).

Report the use of medications such as selective serotonin reuptake inhibitors (SSRIs) to your practitioner.

5. In addition, caffeine, nicotine, and alcohol must be abstained from for 2 days before the sample collection.

Additional guidelines for women:

- Do not collect urine during menstruation.

**Collect and send from Monday through Thursday.
Preferably, send the test kit on the day of collection.**

Test procedure:

1. After getting up, first go to the toilet as usual. This first morning urine must not be collected. Maintain your subsequent normal daily routine, such as breakfast, with the exception of fluid intake. This must not exceed 0.3 liters (1 glass of water). Coffee is NOT allowed!
2. The subsequent 2nd morning urine must be collected. Use the midstream urine for the urine sample: Let the urine flow into the toilet for approximately 3 seconds. Then collect approximately 25 ml of urine in the urine cup without interrupting the flow. It should be filled to about half full. Let the rest of the urine run back into the toilet.
3. Remove the urine tube with stabilizer from the shipping container (with hazardous material label).
4. Transfer the urine from the cup into the urine tube containing the stabilizer. The tube should be filled up to the 25 ml mark.
5. Close the urine tube carefully and shake it 2–3 times.

pH measurement:

6. For determining the pH value, remove the pH indicator strip from the resealable bag. Ensure that the indicator does not come into contact with your fingers or any liquids.
7. Open the urine tube. Hold the pH indicator strip between your thumb and index finger and dip it into the urine.
8. The strip will change color depending on the pH value.

Shipping

NL: RPSH Analytic, antwoordnummer 1461, 4460 VS Kamperland

Belgium: RPSH Analytic, drop off at an UPS Access Point

Germany: RPSH Analytic, Veerweg 16, 4493 AS Kamperland – Niederlande (Per Express)

Other countries: please contact us at info@rpsn.nl

Please note: Do not send during the weekend or the day before a national holiday!

NL: Within the Netherlands, send from **Monday to Thursday** (from an orange PostNL postbox).

1. Check whether the postbox has already been emptied.
2. Use a medical postbox if the regular postbox is already empty. These postboxes can be recognised by the logo 'medische post'.
3. Find a medical postbox via the PostNL Location Finder, filter on 'post a letter' and select 'medical' at the bottom.

Scan the QR code for the locations of the medical postboxes:

<https://www.postnl.nl/en/locationfinder/>

Please note: If the test kit is not sent via a medical postbox, but via a PostNL point, it will be treated as a parcel. This may delay delivery and may prevent the test kit from being processed due to the quality of the sample.

Belgium: From Belgium send from **Monday to Wednesday** via UPS.

- After taking the test kit, the patient can drop the sealed return envelope off at one of the UPS Access Points (do NOT deposit it in a Bpost mailbox!)
- Send from Monday to Wednesday (this guarantees that the test kit can be processed before the weekend)
- The cost for the return shipment is paid by RPSH.

The UPS Access Points can be found on the UPS website under 'Locations' at the top right [ups.com/dropoff/?loc=nL_BE]

Please note: The UPS Access Point you select will indicate the latest collection time for 'Standard' service. The test kit must be dropped off before this time to be delivered to RPSH Analytic the next day!

RPSH
Sanitas Humanus